


When you want a boost

If you were exposed to loud sounds, you would seek hearing protection. At the other end of the spectrum though, are difficult to hear, soft sounds. Hearing difficulties for normal listeners is usually related to one of two conditions: Either the sound is softer than you want to hear it normally, or there is competing noise which makes speech harder to understand.

Focus Ear PAL is an elite brand of Personal Sound Amplifier (PSAP). PSAP's are intended to improve a normal hearing condition in difficult situations.

Identifying Your Focus Ear PAL™

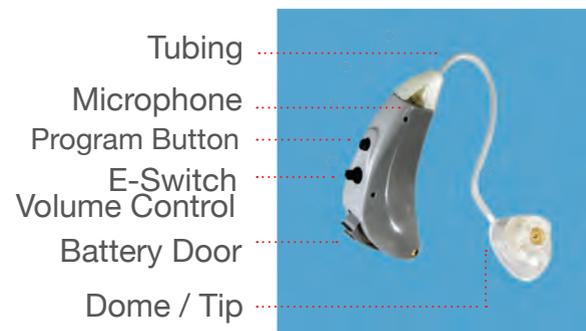

Exclusively from

FOCUSEAR™
PERSONAL SOUND AMPLIFIER

Personal Sound Amplifier

Instruction Guide

www.focusear.com

For customer support please call 888-233-6678

Focus Ear PAL™

Focus Ear allows you to enhance your listening skills....like reading glasses for your ears! A high definition, high fidelity personal sound amplifier with 100% digital technology will amaze you in noisy situations and improve your ability to focus on the important sounds you want to hear.



Call for customer support

1-888-233-6678

Cleaning & Changing Tubes


Use a soft cloth to wipe the PAL after use to remove any debris or moisture. The Cleaning Tool is used to remove debris / wax from the sound tubing and earbud.




1. To remove tubes gently rotate the sound tubing holding the instrument firmly.



2. Pull sound tubing from the instrument by pulling directly opposite from the body of the PAL.

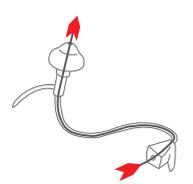

3. To clean, insert the cleaning tool through the nose cone end. Thread all the way through the tubing & earbud.

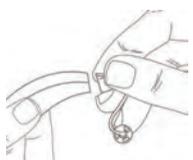

4. To reassemble or change sound tubing, simply line up and push tubing until it snaps into place.

Powering On/Off Your Focus Ear PAL™

PAL uses a hearing aid size 312 battery.

TO TURN ON - With your finger, gently open the door. Insert battery and close the door. Your Focus Ear System is equipped with a power on delay, you will be given several seconds to seat the device comfortably in your ear before it turns on.

When placing the battery in your Focus Ear PAL device, position the battery with the positive (+) flat side up and insert it into the battery compartment section of the battery door. If the battery is inserted incorrectly or upside down, the door will not close. **Do not force the door to close.**

Before insertion, please clean your ear and make sure the battery door is closed properly.

CAUTION: Batteries are dangerous if swallowed! Keep batteries out of the reach of children. Never put batteries in your mouth for any reason they are easily swallowed. If a battery has been swallowed, seek immediate medical attention. For more information call the National Battery Hotline at 202-625-3333.

NOTE: Some irritation may occur due to becoming accustomed to wearing a new instrument. If swelling or redness occurs, discontinue use and visit your Doctor.

Inserting Your Focus Ear PAL™

• Hold the tubing with your thumb and index finger close to the ear dome and place the instrument over the top of your ear. (Fig. A).



Fig. A

• Gently insert the ear dome into your ear canal. (Fig. B)



Fig. B

• Move the retention lock into position by lifting and placing it into the bowl of your ear. The retention lock may be cut off with scissors if it bothers you or is difficult to use. (Fig. C)



Fig. C

VOLUME ADJUSTMENT (Fig. 1 & 2)

Use your thumb and middle finger to hold each end. Use your pointer finger to toggle the E-Switch up/down (Fig. 1). The E-Switch volume control rocks back and forth, always returning to the center position. EACH SHORT PRESS (1 sec.) will raise or lower the volume slightly. Pushing up increases the volume. (Fig. 2)



Fig. 1



Fig. 2

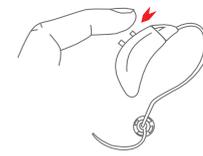


Fig. 3

Volume Adjustment (cont'd)

Pushing down lowers the volume. There are a maximum of ten volume settings. Every time you turn on the PAL, it will start at the last volume you had set.

TO REMOVE - Simply grasp the tubing close to the earbud or earmold and remove from ear canal. Lift instrument from the top of the ear.

TO TURN OFF - To turn your Focus Ear PAL off, simply open the battery door completely. Store in a safe dry place without the battery inserted.

PROGRAMS / MEMORIES (SEPARATE CONTROL)

After inserting the battery, the PAL starts in program one. Pushing the short button (most forward) will change the memory. (Fig. 3) The number of beeps correspond to the program. The programs cycle in sequence 1,2,1,2 etc....

Find the program that is most comfortable for you and your environment.

- Programs:**
- 1** - Speech - Normal, everyday use
 - 2** - Noise - Restaurant, meetings, etc.
 - 3** - Optional Telephone coil